

# Children's Eye Massage Instrument

## 兒童眼部按摩儀

Model : MS-ME

### Features:

- Use it for 15 minutes twice a day to relieve eye fatigue
- Small and portable, foldable, portable, pocket, bag anywhere
- Three hot compress modes: wave heating is used for hot compress, and three modes can be switched by one button
- 55dB silent vibration massage, double engine high frequency vibration massage, as quiet as turning a book
- According to the ergonomics, with latex cotton and milk silk fabric, it can contact like skin and perfectly fit all kinds of facial contour

- 每次使用15分鐘，一天兩次，舒緩眼睛疲勞
- 小巧便攜，可折疊，隨身攜帶，口袋、包包隨便放
- 三種熱敷模式：熱敷採用波浪式加熱，三種模式一鍵切換
- 55dB靜音振動按摩，雙引擎高頻振動按摩，如翻書般安靜
- 根據人體工程學，配合乳膠棉和牛奶絲面料，如肌膚般接觸，完美貼合各類臉部輪廓



### Specifications

Product Model 產品型號	MS-ME
Product Material 產品材質	ABS+ latex cotton ABS+乳膠棉
Power mode 電源方式	USB
Color 顏色	white 白色
Applicable parts 適用部位	eyes 眼部
Massage principle 按摩原理	vibration massage+third gear hot compress 振動按摩+三擋熱敷
Packing size 包裝尺寸	11.5*6.5*7cm
Package weight 包裝重量	159g

### Instructions

調整產品鬆緊帶合適長度後，佩戴於眼部  
開機：長按按鍵2秒，產品開機並振動提醒，開始工作，默認工作時長15分鐘，定時結束後自動關機並振動提醒  
產品工作中，首次默認模式：高溫、中溫、低溫

#### 注意事項

使用前，請先取下佩戴的隱形眼鏡、發卡、髮飾等，避免按摩時不適。請閉目使用本產品  
產品有發熱表面，對熱不敏感的用戶使用時必須注意；如果感到眼睛或眼睛周圍出現疼痛或灼熱時，請停止使用  
產品包裝設有簡易防震設施，適合航空、鐵路、公路及輪船運輸，應避免雨或雪淋濺，倒置和碰撞  
清潔時應在關機的狀態下進行，如需清潔外觀，應用酒精棉花清潔，避免用水清潔。當出現故障時請專業技術人員維修，非專業人士不可自行拆裝產品  
6歲以下兒童禁止使用，6-12歲兒童必須在父母或有監督能力的人指導和看護下使用